

Bone Expander Kit



Manufactured by:

ITL Dental
31 Peters Canyon
Irvine, California 92606 USA
800.277.0073 • 949.223.8950
Fax: 949.223.8960

EU Representative:
HealthLink Europe Services BV
De Twelling 20-22
5215 MC 'sHertogenbosch
The Netherlands

Bone Expander Kit

Bone expander drills are an alternative to Osteotomes for the expansion and condensing of the atrophic mandible and maxilla. They are also an alternative to the maxillary sinus elevation technique. Bone expanders improve the clinical success by improving stability, maintaining bone density and increasing fixation.

Bone expanders can be used with a low speed hand piece or a ratchet wrench.

Kit Contents:

Bone Expanders:	BEK 6
Ratchet Wrench:	1
Pilot Drill:	1
Ratchet Wrench Extender:	1
Hand piece latch adaptor:	1
Manual knob, for finger usage:	1

Sterilization:

This kit may be autoclaved. Follow the manufacturer of the sterilizer's instruction for use while observing the following cautions:

Caution:

- **Do not** exceed 275° (135°C)
- **Do not** submerge in any solution
- **Do not** use ultrasonic devices
- **Do not** autoclave for extended periods of time (i.e. overnight). Extended autoclaving should not be confused with repeated autoclaving. Repeated autoclaving will not harm the product.

Warranty:

The Bone Expander kit parts are warranted to be free from defects in material and workmanship for a period of one (1) year from the date of purchase. The warranty will not apply if the product has been subject to abuse or misuse, nor does it apply to normal wear. Sole liability under this warranty is limited to repair or replacement, at the option of the manufacturer of the defective product. No other warranties are expressed or implied. Any defective product must be returned to the factory (freight pre-paid) with proof of purchase enclosed.

CE