

# Implant Instrumentation

## SURGICAL DRILLS

The step drills are coated to improve the visualization of the score lines and to increase cutting lubricity. They are available in 7 diameters. The first 4mm is the smaller diameter stepping up to the larger diameter.



## Drill Sequence Chart

Drill Code*	Drill Code No Hub	Size	Color	Platform/Implant Diameter					
				3.5/3.2	3.5/3.7	3.5/4.2	4.5/4.7	4.5/5.2	5.7/5.7
SD2320	SDS2320	2.3/2.0	White	X	X	X	X	X	X
SD2823	SDS2823	2.8/2.3	Grey	X	X	X	X	X	X
SD3428	SDS3428	3.4/2.8	Green		X	X	X	X	X
SD3834	SDS3834	3.8/3.4	Pink			X	X	X	X
SD4438	SDS4438	4.4/3.8	Purple				X	X	X
SD4844	SDS4844	4.8/4.4	Blue					X	X
SD5448	SDS5448	5.4/4.8	Yellow						X

Last drill is used in dense bone situations. For soft bone, delete last drill.

\* For use with drill stops.

(Above photos not to scale)

Implant Instrumentation  
continued on next page...